



# Sani-Cloth® AF is a two in one solution ideal for cleaning and disinfecting.

- + **Ready to use:** pre-dosed, alcohol free, low linting disposable cleaning and disinfection wipes, requiring no mixing or measuring.
- + **Proven efficacy:** effective against Coronavirus and Norovirus in as little as 30 seconds.
- + **Safe to use:** alcohol free, bleach free and PHMB free, causing no skin irritation or sensitisation.
- + **Improved efficiencies:** one product to clean and disinfect, providing you with consistent and reliable results.
- + **Maximum flexibility:** suitable for any healthcare environment available in flow wraps, canisters and buckets.
- + **Optimal protection:** a quaternary ammonium based formulation which can be used on non-invasive medical devices; non-porous hard surfaces.
- + **Cost effective:** using the same solution for cleaning and disinfecting saves you time and money.



## Product Efficacy

SANI-CLOTH® AF	TEST METHOD	ORGANISM	CONTACT TIME*	
<b>Bacteria</b>	EN 16615	Pseudomonas aeruginosa	60 sec (c)	2 min (d)
	EN 16615	Staphylococcus aureus	60 sec (c)	2 min (d)
	EN 16615	Enterococcus hirae	60 sec (c)	2 min (d)
	EN 13727	Pseudomonas aeruginosa	30 sec (c)	
	EN 13727	Staphylococcus aureus	30 sec (c)	
	EN 13727	Enterococcus hirae	30 sec (c)	
<b>Yeasts</b>	EN 16615	Candida albicans	60 sec (c)	2 min (d)
	EN 13624	Candida albicans	30 sec (c)	
	EN 13624	Candida auris	30 sec (c)	
<b>Enveloped Virus</b>	EN 14476	Vaccinia Virus	30 sec (c)	2 min (d)
	EN 14476	Corona,HIV, Hep B & C, SARS, Flu	30 sec (c)	2 min (d)
<b>Non-enveloped Virus</b>	EN 14476	Norovirus	30 sec (c)	60 sec (d)
<b>TB</b>	EN 14348	Mycobacterium terrae	2 min (c)	

\*Clean Conditions (c) Dirty Conditions (d)

## Point of Use Accessories



### Wall bracket for flow wraps

Product Code: **A003GRN**

Made with antimicrobial properties proven to reduce the growth of bacteria by up to 99.99%.



### Bracket for canisters

Bracket: **A001GRN**

Canister brackets affix securely to the wall, holding the canister in place at its location.



### Pole Adapter

Product Code: **A002POL**

Pole brackets make it possible to affix the canister brackets on to portable equipment such as BP stands.



### I Am Clean Sticky Notes

Product Code: **A005SNT**

Recyclable sticky notes which can be used to easily and effectively identify equipment or surfaces that have been cleaned.



### Educational wall charts and signage

Product evaluations, audits, educational wall charts & signage, that can be personalised for your specific needs.

## Range Details

	REORDER NO	WIPE COUNT	WIPE SIZE	QTY PER CASE	CLASSIFICATION
225ct Bucket	EAF1110	225	270mm x 267mm	1	Medical Device Class IIa
200ct Flow Wrap	EAF1130	200	267mm x 210mm	6	Medical Device Class IIa
100ct Large Canister	EAF1120	100	200mm x 195mm	6	Medical Device Class IIa
40ct Flow Wrap	EAF1140	40	200mm x 200mm	24	Medical Device Class IIa

Other formulations available:



Find out more about how our Layered Approach to Infection Prevention can help you.

PDI EMEA, Pywell Road, Willowbrook Est. Industrial Estate, Corby - Northants, NN17 5XJ, UK

+44 (0) 8081 697 945

contact@pdi-emea.com

wearepdi-intl.com

PDI International

PDI\_Intl

**PDI**  
BE THE DIFFERENCE™