

# Sani-Cloth® AF: Instructions for Use

## Two in one cleaning & disinfectant alcohol-free wipes

Please follow local healthcare policy for personal protective equipment, hand hygiene & waste disposal guidelines



### Preparation



### Dispense




- + Dispense the wipe at a 45° angle, pulling away from you.
- + Close the lid when not in use.

### Clean



- + If the surface is visibly soiled, clean the surface following an S-shaped motion from top to bottom.
- + Use additional wipes as needed.
- + Use a fresh wipe for each new surface.

### Disinfect

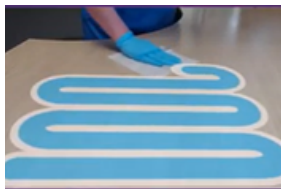

- + Using a fresh wipe, disinfect the surface in an S-shaped motion from top to bottom, avoiding cross-contamination.
- + Use a fresh wipe for each new surface.

### Contact Time



- + Ensure the surface remains visibly wet for 2 minutes for complete disinfection.
- + Use an additional wipe if necessary.
- + Allow to air dry.

### Disposal



- + Dispose of wipe after a single use.
- + Packaging is not recyclable



### Other information:

- + Store upright in a cool dry place
- + Avoid contact with eyes, skin & clothes
- + Refer to expiry date and POA on packaging



For more information, write to us: [contact@pdi-emea.com](mailto:contact@pdi-emea.com)

+ 44 (0) 8081 697 945    [contact@pdi-emea.com](mailto:contact@pdi-emea.com)    [wearepdi-intl.com](http://wearepdi-intl.com)     PDI International

