

eFONA* in CICO situation

All components in one set

Compact pack size




Surgicric II

Set for surgical cricothyrotomy according to the classical surgical technique

/ Surgicric II

Surgicric II is a set for surgical cricothyrotomy according to the classical surgical technique. The set enables securing the airway in case of upper airway obstruction, respectively as "Ultima Ratio" if all other attempts to ventilate the patient failed. The special feature of Surgicric II is the combination of tube and dilator. The pre-inserted dilator in the tube allows for a rapid, easy procedure. Packed in a specific sterile pouch, the set is providing clear overview of all components.

Use

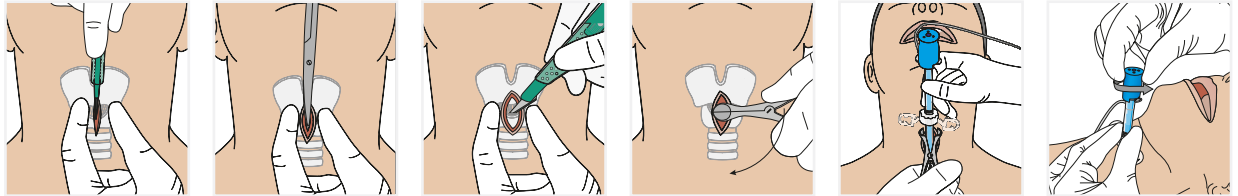

1. Vertical skin incision
2. Blunt dissection
3. Horizontal incision through the cricothyroid membrane
4. Insert speculum and turn by 90°
5. Insert tube via the speculum into the trachea
6. Unlock and remove the dilator

Features

Scissors

- For blunt dissection up to the cricothyroid membrane

Speculum and Tracheal Hook

- To hold the incision open

Dilator

- Soft tip to prevent injury of the posterior tracheal wall
- Smooth transition between dilator and tube tip
- Locking mechanism reduces the risk of accidental dislocation of the dilator from the tube

6.0 mm I.D. Tube

- Cuff for protection against aspiration
- Moveable flange
- Atraumatic, rounded tube tip

Extension Tubing

- With suction possibility ≤ 16 Fr

Necktape

- Safe fixation of the tube



Content

- 1 Scalpel # 11
- 2 Syringe 10 ml
- 3 6.0 mm I.D. tube with cuff and dilator
- 4 Necktape
- 5 Extension tubing
- 6 Tracheal hook
- 7 Scissors
- 8 Speculum

Robust packaging and compact pack size to fit emergency bags, backpacks, crash carts, etc.

Pack size:

31.5 cm (L) x 15.2 cm (W) x 5.0 cm (H)

Unfolded size / sterile field:

56 cm (L) x 39 cm (W)

Order information

Surgicric II / For single use, sterile

REF	Box
30-08-117-1	1

Additional information



Application video



Flyer cricothyrotomy



Annotated bibliography

* emergency Front Of Neck Access

This product is manufactured without the use of natural rubber latex, unless otherwise specified.
This product does not contain phthalates which require labelling according to CLP Regulation (EC) 1272/2008.